



Case Report:

Nd: Yap laser - ProDeep® - chin scar treatment.

Dr. Hugues Cartier, Dermatologist
Arras, France

Treatment:

This is a treatment report performed by the dermatologist Dr. Hugues Cartier, from Arras, France, using the Etherea MX® platform, with the ProDeep® handpiece.

- ▶ The unaesthetic scars can arise after burns, injuries, post-surgery or even after more severe cases of acne. The scars' location can directly affect a patient's quality of life.
- ▶ Among the treatments described in the literature to improve the scars appearance on the face are micro needling and ablative and non-ablative fractional LASERS.
- ▶ The ProDeep® handpiece, compatible with the ETHEREA-MX® and ZYE® platforms, works with a 1340nm wavelength – Nd:Yap. It is a non-ablative fractional laser, with small water absorption, being deeper than others. Allows for the treatment of recent and old scars, with no phototype restrictions and with little or no downtime.
- ▶ Dr Cartier held 2 sessions, with the parameters described below.

HANDPIECE
ProDeep®

SPOT
8mm - 100mtz

PULSE TIME
5ms

FLUENCE
110mJ/mtz

Check out the result:



Before and after - 2 sessions